



HU-FRIEDY KEY OPINION LEADER


DR. GAETANO PAOLONE AND DR. SALVATORE SCOLAVINO
HU-FRIEDY 3SESSENTIAL KIT BY WERESTORE.IT



WeRestore .it

WeRestore.it is a project founded in 2016 by Gaetano Paolone and Salvatore Scolavino. WeRestore.it aims for excellence in adhesive direct and indirect restorative dentistry. Both Salvatore and Gaetano are active members of the Italian Academy of Restorative Dentistry. They wrote a book about direct posterior restorations edited by Quintessence Int. Publishing. Gaetano is adjunct professor of Restorative Dentistry at Vita e Salute University, San Raffaele, Milan. They lecture all around the world in restorative dentistry and run master courses in the WeRestore.it course center in Rome.


www.werestore.it*

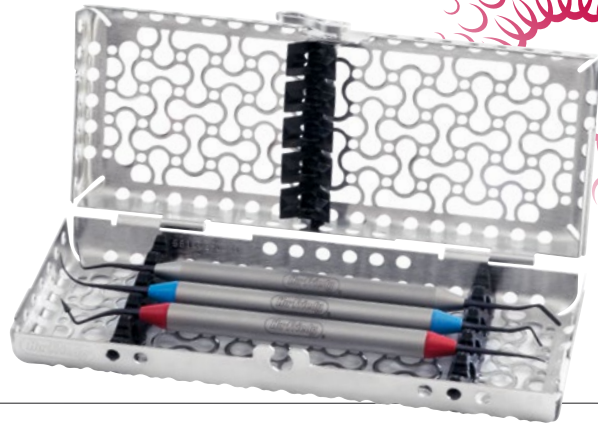

THE ESSENTIAL RESTORATIVE KIT INCLUDING "PLUG&SCULPT" INSTRUMENT

The Hu-Friedy **3SESSENTIAL KIT** by Werestore.it is the most straightforward kit of restorative ever made, with just **3 essential instruments at once: Anterior, Posterior and Spatula**, from the Hu-Friedy Black Line Collection. The three instruments are synergic and interchangeable and the word "Essential" refers to the easiness of all restorative procedures focused on the essential steps to create high esthetic composite restorations while using this kit.

The revolutionary concept of "Plug&Sculpt" of the Posterior is very simple: one tip is used to plug the composite into the cavity and the other one can sculpt composite in additive and subtractive modelling techniques. The Anterior, instead, is called by the doctors "solid brush" since it models and spreads the composite very easily, just like a brush.

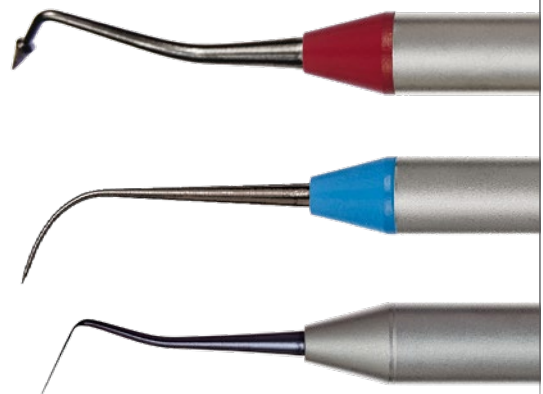

Visit us online at www.hu-friedy.eu

©2018 Hu-Friedy Mfg. Co., LLC. All rights reserved.
*WeRestore.it is not owned by or affiliated with Hu-Friedy.

How the best perform






3SSENTIAL KIT BY WERESTORE.IT THE INSTRUMENTS



TNCIPAAX
Anterior

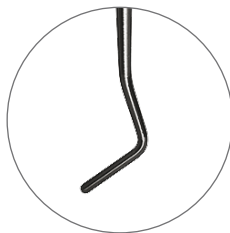

TNEXBKRIK
Posterior



CVKOTIX
Spatula

FEATURES AND BENEFITS

- **Essential** method: having only three instruments makes restorative procedures straightforward
- **Plug&Sculpt** concept: The **Posterior** can plug and sculpt the composite by using its two different tips
- **Solid brush**: The **Anterior** models and spreads the composite easily, just like a brush
- **Different colours**: each colour helps to easily identify the instrument:
RED for Anterior, BLUE for Posterior and GREY for Spatula
- **Black Line Collection instruments**: smooth surface reduces stickiness to composite material and the black colour has more contrast to the dental hard and soft tissues.
- **Ergonomics** and **visibility** improved: allow dental clinicians to perform at their best



3SSENTIAL KIT BY WERESTORE.IT



HOW TO DO?

ANTERIOR




ANTERIOR, TIP #1

It allows you to apply uncured composite material to sculpt and spread it like a brush. It's called "solid brush". Easy to use also to model composite material in Class V restorations.

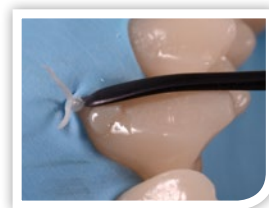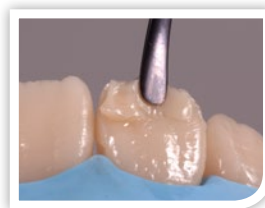

ANTERIOR, TIP #2

Just turn and use to define mamelon structures in dentin.


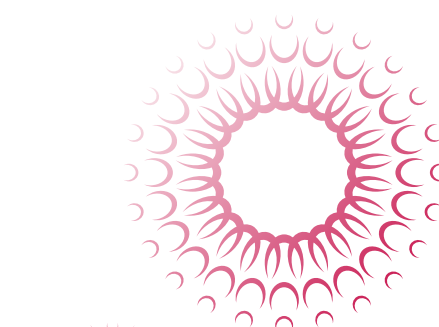


POSTERIOR






POSTERIOR, TIP #1

To sculpt uncured composite material in additive and subtractive modeling techniques.

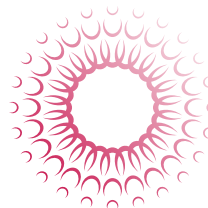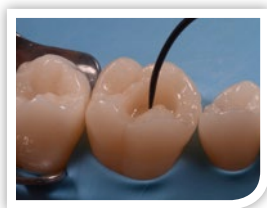

POSTERIOR, TIP #2

To plug uncured composite material into the cavity for the centripetal build-up technique (building interproximal wall to move from a class II to a class I).

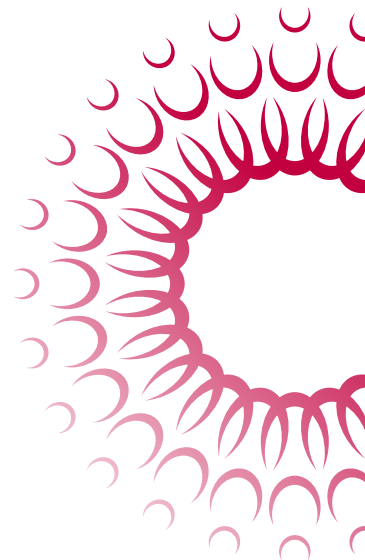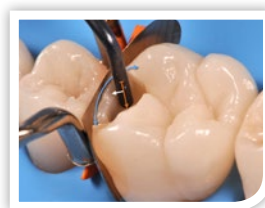
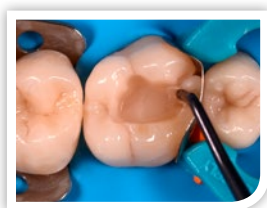


SPATULA




SPATULA, TIP #1 & #2

Thin and cutting spatula design to take uncured composite material and apply it into anterior and in posterior cavities (and then to sculpt it in class II during the centripetal build-up technique).

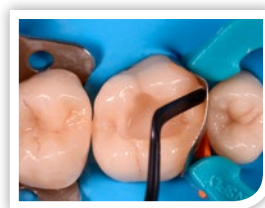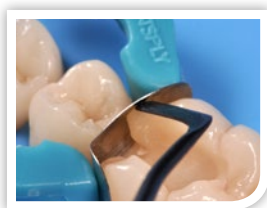

3SSENTIAL KIT BY WERESTORE.IT

THE KIT

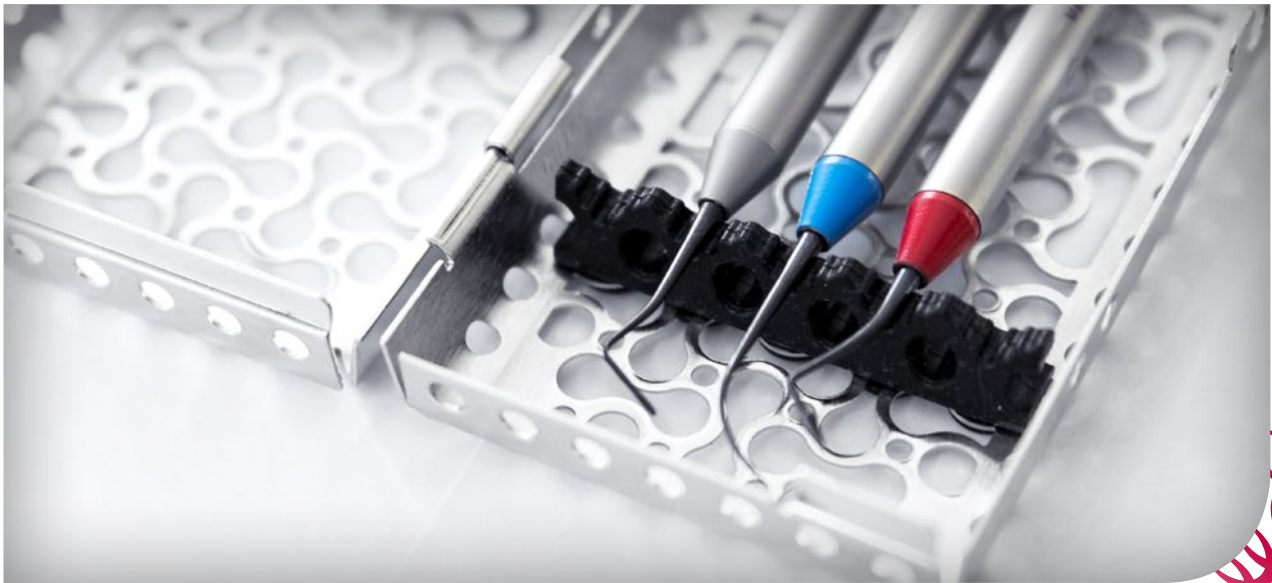

THE 3SSENTIAL KIT BY WERESTORE.IT

PARTCODE **THE 3SSENTIAL KIT INCLUDES:**

- RESTOREITDIN**
- IMS Infinity Cassette 1/4 DIN 5 instruments Black
 - 3 instruments (Spatula, Posterior, Anterior)

PARTCODE **DESCRIPTION**

- | | |
|-------------------|--|
| CVKOT1X | XTS PFI Essential WeRestore.it Spatula |
| TNEXBKRI1X | XTS PFI Essential WeRestore.it Posterior |
| TNCIPA0X | XTS PFI Essential WeRestore.it Anterior |

IMS: WHY USING A IMS INFINITY CASSETTE SYSTEM?

By standardising your instruments with IMS, additional time and revenue is generated while extending the useful life of your Hu-Friedy instruments.

Dental professionals state that by implementing IMS, they save 5 to 10 minutes per procedure allowing the staff to spend more time on value-added activities.

In addition, safer transportation of soiled instruments reduces risk of injury and exposure to blood borne pathogens.

